



CIRCUMCISION DEVICE





Application:

Used in male circumcision for both adults and children in Urology, general surgery, plastic surgery, dermatology.

Product Description:

Circumcision devices have been developed to shorten the operative time, simplify techniques, and improve safety and cosmetic outcomes. The devices generally aim to crush the foreskin while simultaneously creating hemostasis, the foreskin is then excised or allowed to slough off

Features:

- Single use, avoid cross-infection
- Minimal tissue injury
- Fewer postoperative complications
- Shorter operative time, lower blood loss volume
- Time-efficient and safe male circumcision technique
- Faster recovery and nice cutting edge
- Applied both children and adults

Direction of Use



Step 1:

Rotate the adjust knob anticlockwise and take out the anvil

Step 2:

Pull up the foreskin with hemostatic forceps, insert the anvil into the foreskin and cover the glans, leave more frenulum to avoid the frenulum hurt.



Step 3:

Tie foreskin to the central of anvil, take out the protector cover, connect the central of anvil to stapler, load the adjust knob and tight clockwise till the red indicator can be touched, the stapler is ready for firing.

Step 4:

Take out the safety.





Step 5:

Fire the stapler till a clear “click” shows the suture is done. Keep in pressing for 10 seconds to avoid hemorrhage.

Step 6:

Release the moving handle, turn the adjust knob anticlockwise 2-5 circles and push the adjust knob, it will separate the stapler and anvil, divide the foreskin with anvil. (if the foreskin is not cut completely, can use scissor to cut)



Available Sizes:

12, 15, 18, 22, 26, 30, 32, 34

JUPITER HEALTH CARE